

GuidedSMILE Case Submission Work Flow

What to send

Dentate

- DICOM Uploaded to www.roedentallab.com or Disc with mailed materials
 - Multi-file, uncompressed data in a zipped folder

Edentulous

- DICOM Uploaded - verified denture using Dual-Scan Technique with radiopaque markers
 - Scan of patient wearing scan appliance in occlusion
 - Scan of scan appliance resting on foam or cotton rolls

With All Cases:

- GuidedSMILE Work Authorization completed – can be completed on line with uploaded DICOM
- Photographs: Full Smile / Lips at rest (repose)
- Master Surgical Guide Agreement signed by placing doctor (1 time)
- Surgery Date – minimum 16 laboratory days
 - 5 days in lab for diagnostic work-up once all materials are received
 - 10 days in lab for guide fabrication – then ship to office
- Master casts (digital for dentate patients with stable bites)

Guide Planning

- Within 5 business days of receiving all the materials you will receive a phone call or email to schedule the on line meeting to plan the case. Once the meeting is complete, ROE prepares the case over the following 10 lab days.

Ship to:

ROE Dental Laboratory
7165 E. Pleasant Valley Rd
Independence, OH 44131

800 228 6663 CT Department
www.roedentallab.com
info@roedentallab.com

UPS label ROE web site

