

GuidedSMILE CHROME Records



<p>↑ Dentate ↓ Dentate</p>	<p>↑ or ↓ Denture ↑ & ↓ Denture (both arches)</p>	<p>Ill-fitting denture or modification needed ↑ or ↓ Denture ↑ & ↓ Denture (both arches)</p>
<p>Question: does the patient have 70% or more metal-based restorations in the arch of interest = must send models to ROE to make scan appliance</p>	<p>Denture(s) must exhibit excellent fit. If not perform a hard reline (never soft, remove soft if present). If tooth position or midline are slightly off, relay changes on Rx. Perform the Dual Scan Technique using SureMarks, order from ROE. If opening bite more than 2mm, fabricate a new denture.</p>	<p>If dentures are not ideal and require more than subtle changes, fabricate a new full denture. Use SureMarks for the dual scan technique. This denture can be used as a 'just-in-case denture for surgery'.</p>
<p><u>Records necessary to begin the case</u></p> <ul style="list-style-type: none"> • Full smile & Front retracted photograph • Optional profile photo Class II & III • CBCT patient open biting on cotton rolls • Full arch analog or digital impressions • CO or CR bite • Full GuidedSMILE Rx completed when uploading DICOM 	<p><u>Records necessary to begin the case</u> <i>Denture in <u>One</u> arch</i></p> <ul style="list-style-type: none"> • Full smile & Front retracted photograph • Optional profile photo Class II & III • CBCT w/ patient wearing Scan Appliances, biting in occlusion, no cotton rolls or bite fork. • Full arch analog or digital impressions • CO or CR bite • Full GuidedSMILE Rx completed when uploading DICOM • Dual Scan Technique SureMarks <p><u>Records necessary to begin the case</u> <i>Denture in <u>Both</u> arch</i></p> <ul style="list-style-type: none"> • Full smile & Front retracted photograph • Optional profile photo Class II & III • CBCT w/ patient wearing Scan Appliances, biting in occlusion, no cotton rolls or bite fork. Dual Scan Technique SureMarks • No impressions or bite registrations • Full GuidedSMILE Rx completed when uploading DICOM 	<p><u>Records necessary to begin the case</u> <i>Denture in <u>One</u> arch</i></p> <ul style="list-style-type: none"> • Full smile & Front retracted photograph • Optional profile photo Class II & III • CBCT w/ patient wearing Scan Appliances, biting in occlusion, no cotton rolls or bite fork. • Full arch analog or digital impressions • CO or CR bite • Full GuidedSMILE Rx completed when uploading DICOM • Dual Scan Technique SureMarks <p><u>Records necessary to begin the case</u> <i>Denture in <u>Both</u> arch</i></p> <ul style="list-style-type: none"> • Full smile & Front retracted photograph • Optional profile photo Class II & III • CBCT w/ patient wearing Scan Appliances, biting in occlusion, no cotton rolls or bite fork. Dual Scan Technique SureMarks • No impressions or bite registrations • Full GuidedSMILE Rx completed when uploading DICOM