


## Sterilization Steps:

1. Place abutments, screws, parts, etc. in germicidal tray. Be sure to remove residues from all parts.
2. Using a neutral germicidal agent, bathe the components for the time period recommended by the manufacturer.
3. Remove all components from the germicide bath and rinse the components 3 times using distilled water. Use a lint-free cloth to dry all components.
4. Using a high level disinfectant solution soak parts for the requirement amount of time.
5. Remove all parts from the disinfectant solution and rinse 3 times using distilled water.
6. Air dry all parts and immediately insert each part individually into a autoclavable safe packaging that is suitable for steam sterilization.
7. Autoclave all parts for the duration of 7 minutes at 132 °C or for the duration of 20 minutes at 121°C.
8. Store all sterilized parts at room temperature in a dry and dust - free room.

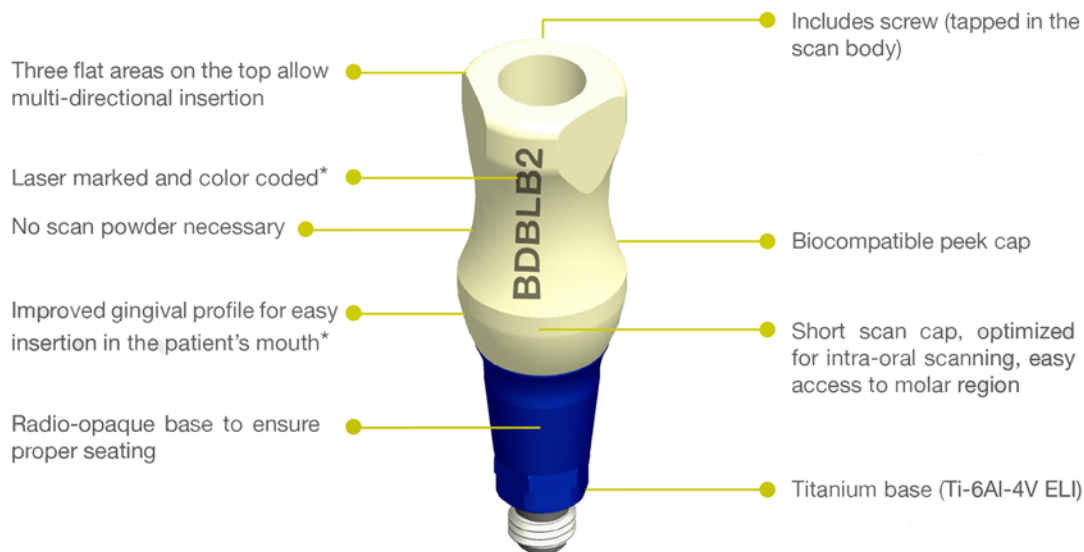

\* Only Biodenta platforms are color coded.

Tax and shipping not included. OEM parts could increase cost. Prices are subject to change without notice.